



Data Transfer Instructions

Application Note

CAUTION: Refer to the Matrix 715 and Matrix 800 User Manuals for warnings, safety information, labeling and detailed operating instructions. This Application Note is only a complement to the User Manuals. If you are currently a Matrix 715 owner and intend to transfer your patient files to the Matrix 800, you must follow these instructions.

Plan the Transfer

- Check the number of tests on the Matrix 715 hard drive – Select F3 and read the number of current test found adjacent to the F3 icon.
- For approximately 20K tests, the transfer will take 5-20 minutes, depending on the hardware configuration of your instrument.

Back Up the Matrix 715

- From the Matrix 715, select F4 and perform a new back to the CD /R-W disk.

Restore to the Matrix 800


- Install the Matrix 715 back up CD disk into the Matrix 800 drive.
- Select F4 and perform a Database Import / Merge or Database Restore if there are no patient files currently on the instrument hard drive.
- 20K tests will take 10-35 minutes to restore.

NOTE: During the restore process, the following will occur:

1. The Matrix 715 data will be converted to the Matrix 800 format.
2. If the patient file did not have a Patient ID, one will be numerically generated and assigned to the patient.
3. If there are patient file conflicts, a resolution screen will appear, giving you the option to resolve the issue.
This may occur if there are duplicate patients, duplicate patient IDs, etc.

NOTE: Once converted to the 800 format, this data cannot be restored to the Matrix 715. If the restore should fail, please try it a second time. If issues continue to arise, please contact your local Customer Service organization for assistance.



 Carl Zeiss Meditec, Inc.
5160 Hacienda Drive
Dublin, CA 94568
USA
www.meditec.zeiss.com/matrix



Carl Zeiss Meditec AG
Goeschwitzer Str. 51-52
07745 Jena
Germany
www.meditec.zeiss.com



We make it visible.